


Instructions for Use
MiniQuick Shell

Self-injection aid for use with Genotropin MiniQuick®

MiniQuick Shell is a medical device intended to simplify injection with Genotropin MiniQuick and hide the Genotropin MiniQuick syringe.


The **MiniQuick Shell** is a multiple use single patient device.

The MiniQuick Shell is exclusively used with the Genotropin MiniQuick syringe.

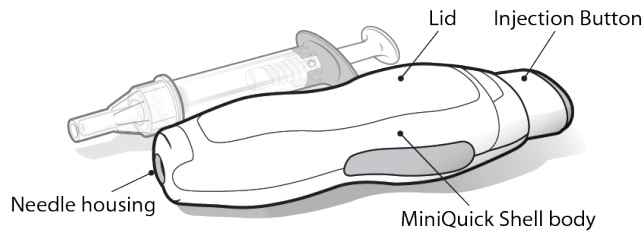


1. Things you need

- A Genotropin MiniQuick syringe (prepared according to MiniQuick instructions with powder and liquid mixed).
- A MiniQuick Shell.
- A suitable sharps container.

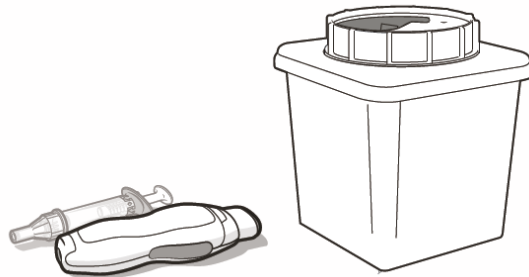


2. Open lid

- Open the lid.

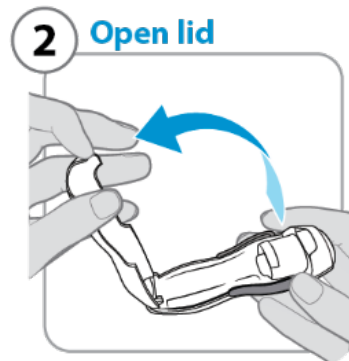


3. Check injection button

- Ensure injection button is out fully before you insert syringe.

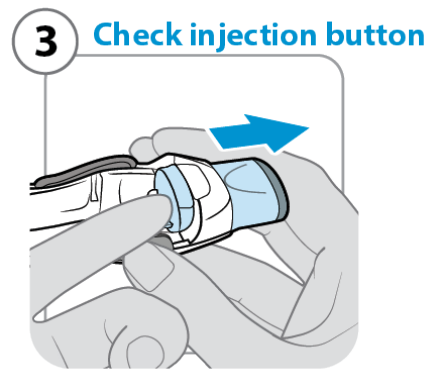


4. Dispose of needle cap

- Pull off the outer needle cap and throw it away (dispose of).
It is not needed again.

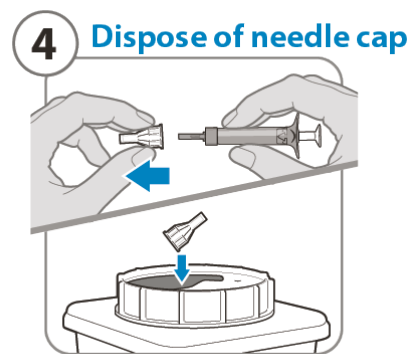


5. Insert syringe

- Place the syringe in the MiniQuick Shell.

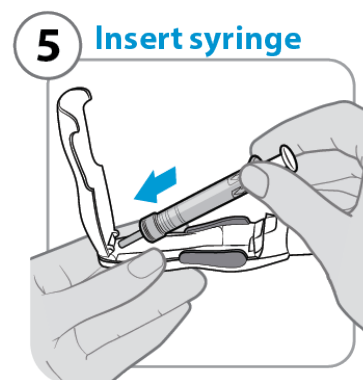


6. Close lid

- Ensure the inner needle cap is through the needle housing.
- Close the lid.

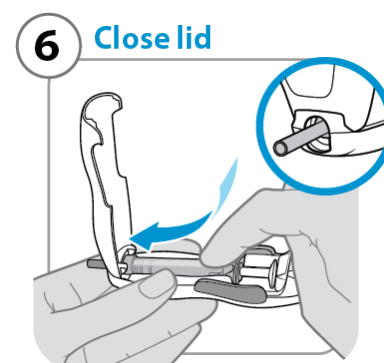


7. Dispose of needle cap

- Pull off the inner needle cap and throw it away (dispose of). It is not needed again.

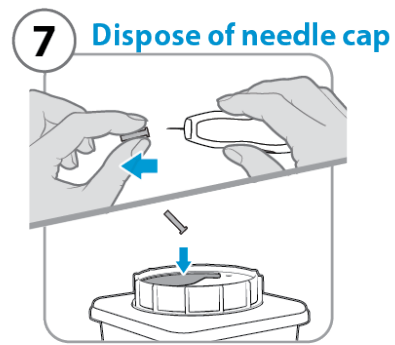


8. Insert needle

- Pinch a fold of skin at injection site.
- Push the needle into the skinfold, as shown.

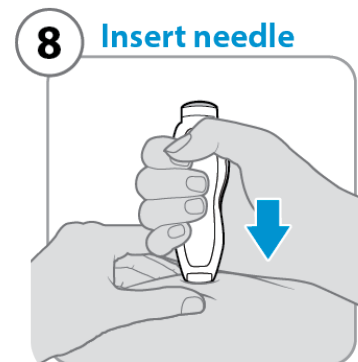


9. Inject medicine

- Press the injection button in as far as possible to inject all the medicine in the Genotropin MiniQuick syringe.
- Hold the button down fully and wait for a few seconds to allow all the medicine to be injected.

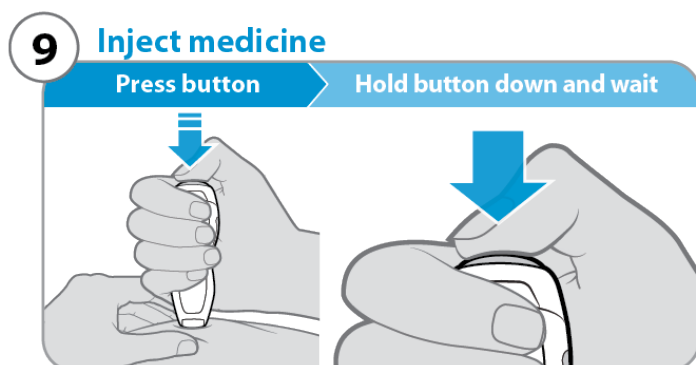


10. Withdraw needle

- Withdraw the needle slowly from the skin at the same angle that it was inserted.

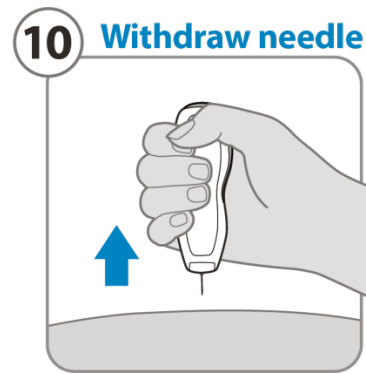

11. Remove syringe

- Ensure injection button is pushed out.
 - Remove the syringe from the MiniQuick Shell.
- Note:** Do not attempt to recap the needle.

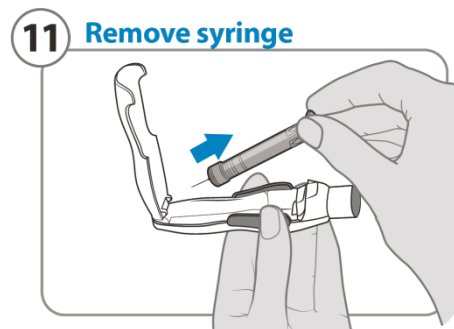

12. Dispose of syringe

- Put your used syringe in a suitable sharps disposal container right away after use.



Questions and Answers

How long can I use my MiniQuick Shell for?

The expected lifetime of the MiniQuick Shell is at least 2 years starting from the date of first use of the MiniQuick Shell.

Do not use the MiniQuick Shell if there is obvious damage.

Do not use the MiniQuick Shell if the injection button does not work or the lid comes apart.

How should I clean my MiniQuick Shell?

Clean the MiniQuick Shell with a damp cloth.

How should I store my MiniQuick Shell?

Store in a dry clean place and keep out of the reach of children.

Further Information








If there is anything you do not understand or cannot do, or if you have any questions about your dose or treatment with GENOTROPIN, contact your healthcare provider.

For questions about the preparation of the Genotropin MiniQuick syringe refer to the Genotropin MiniQuick leaflet or contact your healthcare provider.

Any serious incident that has occurred in relation to the MiniQuick Shell should be reported to Pfizer Australia Pty Ltd at www.pfizermedinfo.com.au and/or the Therapeutic Goods Administration at <https://www.tga.gov.au/reporting-problems>

Date of preparation: March 2022

Symbols

Symbol	Explanation of symbols on packaging
	Marking for conformity with the European requirements
	Date of Manufacture
	Manufactured by: Pfizer Manufacturing Belgium NV Rijksweg 12, B-2870 Puurs, Belgium
	Batch Code
	Medical Device
	Single patient-Multiple use
	Consult Instructions for Use